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DECLARATION OF CE CONFORMITY

The manufacturer: FARO S.p.A. - Via Faro, 150 - 20060 Ornago (Milano) ITALY. declares under its own responsibility that following product: Product name: Dental Turbine - Mod: Trilogy - It is factory new in accordance with the provisions of art.6 para 2 of D.L. 626/94. It fully conforms to the following European directives and national acceptances: 93/42/CEE (medical devices directive) Class which the equipment belongs to: IIa It conforms to the following standards: - ISO 7785-1 - CEI EN 60601-1.

SYMBOLS USED IN THIS MANUAL

Danger - Paragraphs marked with this symbol contain instructions to be followed carefully to avoid damage to the device, the operator and patient. **Warning** - These instructions advise that much attention must be paid to avoid situations which could damage the device. **Prohibition** - This icon points out what to prevent doing to not damage the device. **Suggestions** - Information for most effective use of the device.

SAFETY STANDARDS

• Follow all the instructions set forth in this manual. • The device is designed to be used only in a dentist's office by medical personnel for performing specific dentistry operations. • The device must be sterilized before each operation. • The device is not supplied sterile and should therefore be sterilized before use. • The sterilization parameters should correspond with those of the Pharmacopoeia Official (ex.: 121°C/134°C). • Never immerse the Turbine in any disinfecting liquid. • The device should always be washed before sterilization. • The device is to be installed by the user on a purposeful power system (Dental Unit) and connected to a cord. • Never press Button when Turbine is turning. • The systems of fastening to the cord meet standards EN 29168. • Only fit in the Turbine chuck Burrs whose shanks meet the requirements of ISO standard 1797. • Make sure the Burr is perfectly seated on the ratchet. • Do not work on the Turbine. Only perform operations illustrated in this manual. For all other problems apply to our Service Department. • While in its packing the equipment will withstand the following environmental conditions for 15 weeks: => Surrounding temperature: from -20°C to +70°C => Relative humidity from 10% to 90%. => Atmospheric pressure from 500mBar to 1060mBar.

FEATURERS

The function of the device is to give movement to a rotating body (for example, a Burr) to allow performance of specific dentistry operations. The Trilogy Dental Turbine is available in the following models: Trilogy quick connection - Trilogy plus quick connection - Trilogy luce quick connection - Trilogy plus luce quick connection

ALL TRILOGY TURBINES HAVE A CONTAMINATED LIQUID RETENTION SYSTEM. • Turbine with Burr change by pressing Button on Turbine head. • Connect with Cord by Quick Connector. • Turbine Midwest.

INSTALLATION

Make sure the package contains the following parts: • Dental Turbine • Pin • Wrench for Valve extraction and Ringnut Cover disassembly • Instruction Manual
OPERATING INSTRUCTIONS MAKE SURE WATER AND AIR ARE CORRECTLY FILTERED AND THAT THE SPECIFICATIONS ARE OBSERVED. REPLACING BURR To replace Burr, press Button on Turbine Head. **PRESS BUTTON ONLY WITH TURBINE STOPPED.**

The Burr tightening system with a Button was conceived to be fully effective during rotation. To avoid damaging locking system, do not attempt to remove Burr without pressing Button.

MAINTENANCE CLEANING • The Turbine can be sterilized with cycles whose parameters are in agreement with the Official Pharmacopoeia. Normally use 121°C and 134°C cycles suitable for sterilization of hollow bodies. • The Dental Turbine should always be sterilized after each patient. • Before sterilizing, the device should always be cleaned on the outside with a brush under running water or with an alcohol-based solution. • The Trilogy Turbine can be disinfected externally with chemical disinfectants (disinfection by spray or with a damp cloth). • Do not leave the Turbine immersed in disinfecting liquids. • The device is not washable in the Heat Disinfectant.

LUBRICATION • Disconnect the Turbine by pulling on the Quick Connector in axial direction. • Insert the Aerosol Can Injector of the lubricant in the Turbine hole up to the Striker, then spray the lubricant. • Unscrew the Aerosol Injector from the Turbine and reconnect the handpiece to the Quick Connector. Run the Turbine without load for a few moments.

ALWAYS LUBRICATE THE TURBINE AFTER STERILIZATION. REPLACING ANTICONTAMINATION VALVE

• Fully unscrew Body "A". • Detach Internal Body "B" from Head Unit. • Screw the supplied wrench into the hole "C" where the Valve is fitted then pull the wrench in the direction of the arrow It

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is absolutely forbidden to use the Turbine without the Anticontamination Valve.

NOZZLE CLEANING

If the small holes from which water and air emerge are obstructed, free the passages by using the purposeful Pin "D" • It is recommended to carry out this task weekly.

CLEANING DUCTS

• Unscrew the Front Ringnut "C" using the wrench supplied. Free the passages with the purposeful Pin "D" • Tighten the Ringnut carefully being careful not to damage the two gaskets.

TROUBLESHOOTING

• The sound of the Turbine has changed, the vibrations are excessive and cutting power is low - Apply to our Service Department • The fiberoptic Turbine does not illuminate. - Check condition of Light Bulb located on Quick Connector and change in accordance with Quick Connector manual as necessary. - In case of problems apply to our Service Department. • No water or spray comes out of Turbine. - Check whether water outlet holes on Turbine are obstructed and clean if necessary. See "Nozzle Cleaning" - Replace Anticontamination Valve - Clean ducts as set fort - In case of problems apply to our Service Department. • The Burr locking system or the rotor is damaged. - Apply to our Service Department.

SPECIFICATIONS TRILOGY

B-type apply part Rotation speed, [rpm] > 340,000 Stall torque 310g Operating air pressure [bar] 2.9 ± 0.1 Air consumption [l/min.] 32

Water consumption, [ml/min.] 80 Noise level [dB [A]] <65 Vibrations [m/s²] <2.5 Dimensions of head [mm] dia. 11.7 / ht 15 dia. 12.5 / ht 15 Weight [g] 200

To have a safety system it is recommended to use the FARO dedicated electronic circuit. TURBINA TRILOGY TIPO "B" - "B" TYPE TRILOGY TURBINE - TURBINE TRILOGY TYPE "B" - TURBINE TRILOGY TYP "B" - TURBINATRILOGY TIPO "B"

Sostituzione lampadina: Svitare il corpo "A" - Replacement of bulb: Unscrew the body "A" - Replacement de l'ampoule: Desserrer le corps "A" - Auswechseln der Glühlampe: Körper "A" aufschrauben - Sustitución de la bombilla: Desenrosar el cuerpo "A" - Per sfilare e inserire la lampadina utilizzare il foro presente sulla plastica - To remove the bulb, use the hole present on the plastic - Pour dégager et mettre en place l'ampoule, passer par le trou présent sur le plastique - Zum Herausnehmen und Einführen der Glühlampe Loch Plastik benutzen - Para extraer y colocar la bombilla utilizar el agujero presente en el plástico

ATTENZIONE: Turbina tipo "B" pressione di esercizio aria bar 2,5 ± 10% ATTENTION: "B" type turbine has a working air pressure of 2.5 bar ± 10% ATENCION: Turbina tipo "B" presión de servicio del aire 2,5 bares ± 10%

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